



# Product Reference Guide

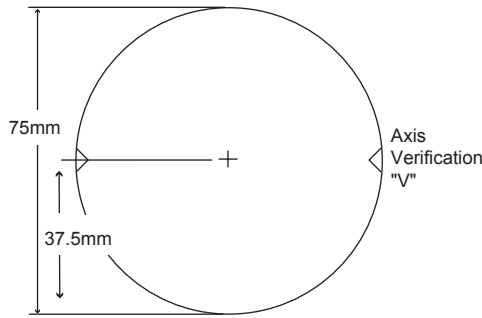

## Base Curve Selection

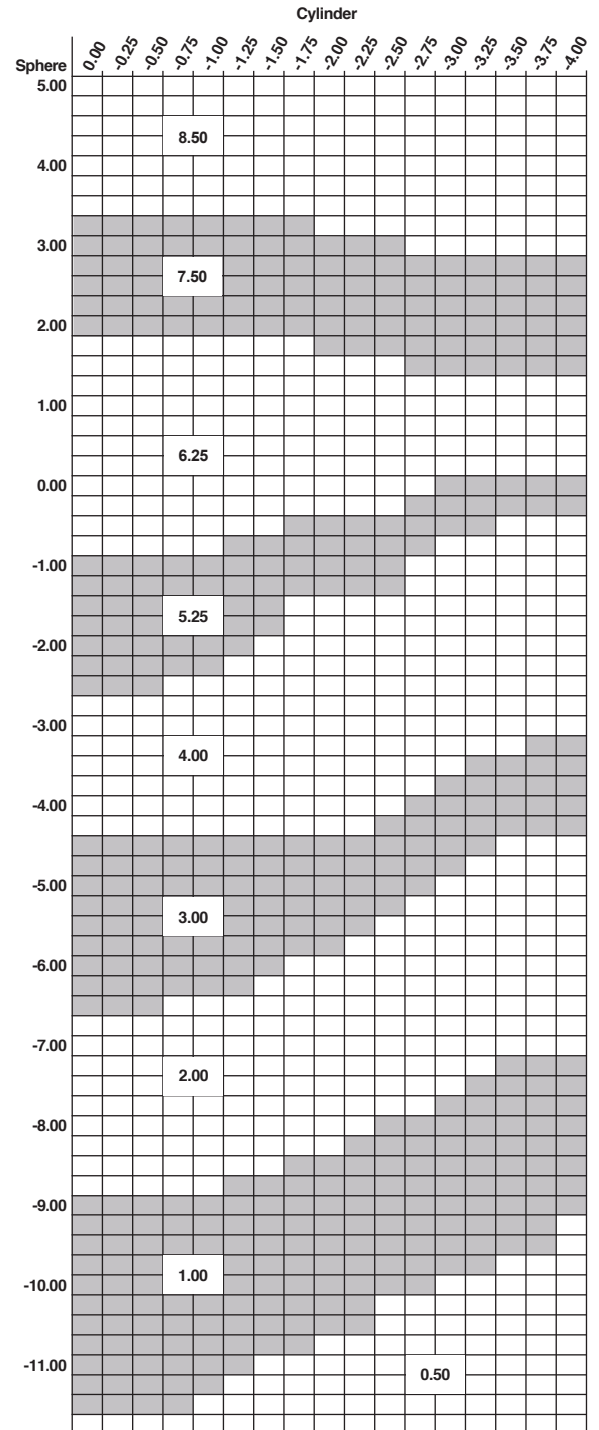

## Polycarbonate


### Coppertone® Prescription Polarized Semi-Finished Single Vision

75mm Polarized Single Vision (HEV Brown, HEV Gray, HEV Green)


Base Curve	True Curve (1.530)	Front Radius	50mm SAG*	Back Curve	Back Radius	CT (mm) Nominal	CT (mm) Minimum	ET (mm) Nominal
0.50	0.45	1177.778	0.265	-6.25	-84.800	9.4	8.9	17.5
1.00	1.13	469.027	0.667	-6.25	-84.800	10.2	9.7	17.4
2.00	2.03	261.084	1.200	-6.25	-84.800	8.0	7.5	14.0
3.00	2.94	180.272	1.742	-6.25	-84.800	9.2	8.7	14.0
4.00	4.07	130.221	2.422	-6.25	-84.800	8.8	8.3	12.0
5.25	5.20	101.923	3.114	-6.25	-84.800	8.6	8.1	10.2
6.25	6.33	83.728	3.819	-6.25	-84.800	8.6	7.6	8.0
7.50	7.46	71.046	4.544	-6.25	-84.800	9.5	8.5	7.5
8.50	8.60	61.628	5.299	-6.25	-84.800	11.0	10.5	7.0

\*Utilized for power calculation only, not for thickness calculation.

# Product Reference Guide

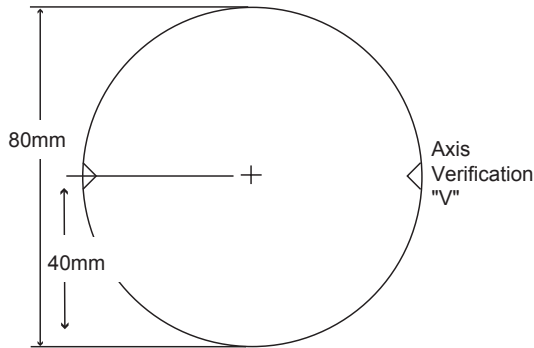

**Base Curve Selection**

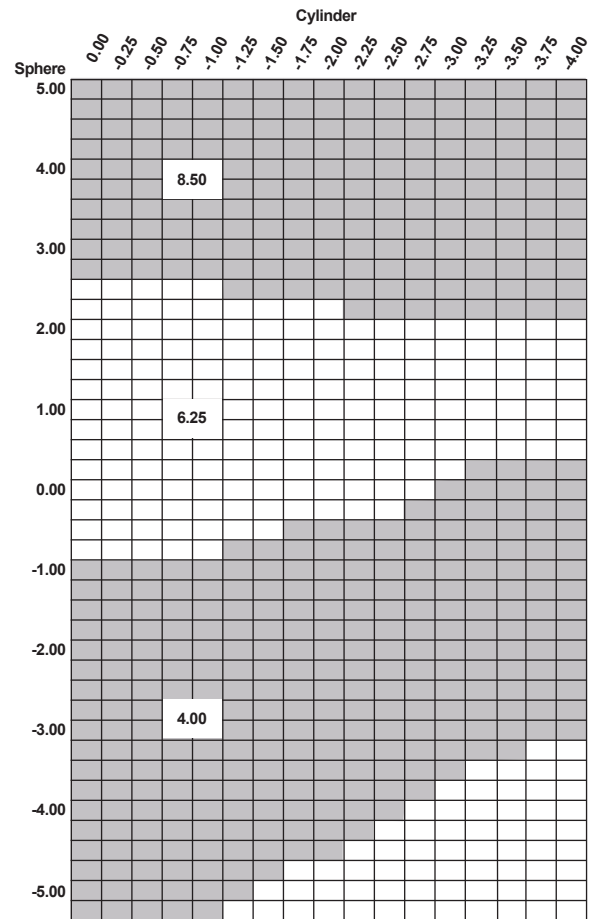

## Polycarbonate


### Coppertone® Prescription Polarized Semi-Finished Single Vision

80mm Polarized Single Vision (HEV Brown, HEV Gray, HEV Green)

Base Curve	True Curve (1.530)	Front Radius	50mm SAG*	Back Curve	Back Radius	CT (mm) Nominal	CT (mm) Minimum	ET (mm) Nominal
4.00	4.07	130.221	2.422	-7.50	-70.67	7.5	7.0	13.7
6.25	6.33	83.728	3.819	-7.50	-70.67	8.6	8.1	10.9
8.50	8.20	64.63	5.031	-7.50	-70.67	10.7	10.2	9.3

\*Utilized for power calculation only, not for thickness calculation.

# Product Reference Guide

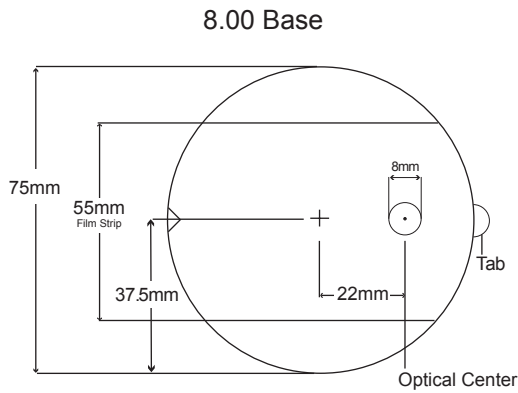

## Polycarbonate


### Coppertone® Polarized Plano Lenses (Plano Uncuts)

75mm Finished Single Vision (HEV Gray, HEV Brown)

Base Curve	Front Curve	Front Radius	CT (mm) Nominal
6.00	5.60	94.64	2.0
8.00*	7.60	69.74	2.0

\*De-centered. The optical center of the 8.00 base is an 8mm diameter circle from the geometric center along a line connecting the geometric center and the center of the tab.

# Product Reference Guide

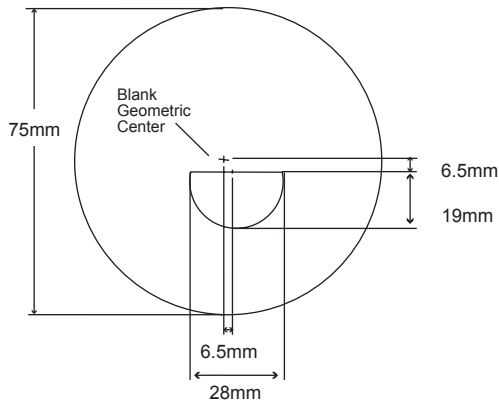

## Polycarbonate

### Coppertone® Prescription Polarized Bifocal


75mm Polarized D28 Bifocal (HEV Brown, HEV Gray, HEV Green)

Base Curve	True Curve (1.530)	Front Radius	50mm SAG*	Back Curve	Back Radius	CT (mm) Nominal	ET (mm) Nominal	Segment Drop	Location Inset
2.00	2.03	261.084	1.200	-6.25	-84.800	8.0	14.0	6.5 D	6.5 In
4.00	4.07	130.221	2.422	-6.25	-84.800	8.8	12.0	6.5 D	6.5 In
6.25	6.33	83.728	3.819	-6.25	-84.800	8.6	8.5	6.5 D	6.5 In
8.50	8.60	61.628	5.299	-7.50	-70.667	9.0	7.0	6.5 D	6.5 In


Base Curve	HEV Brown	HEV Gray	HEV Green
2.00	+1.00 to +3.00	+1.00 to +3.00	+1.00 to +3.00
4.00	+1.00 to +3.00	+1.00 to +3.00	+1.00 to +3.00
6.25	+1.00 to +3.00	+1.00 to +3.00	+1.00 to +3.00
8.50	+1.00 to +3.00	+1.00 to +3.00	+1.00 to +3.00

\*Utilized for power calculation only, not for thickness calculation.


### Base Curve Selection

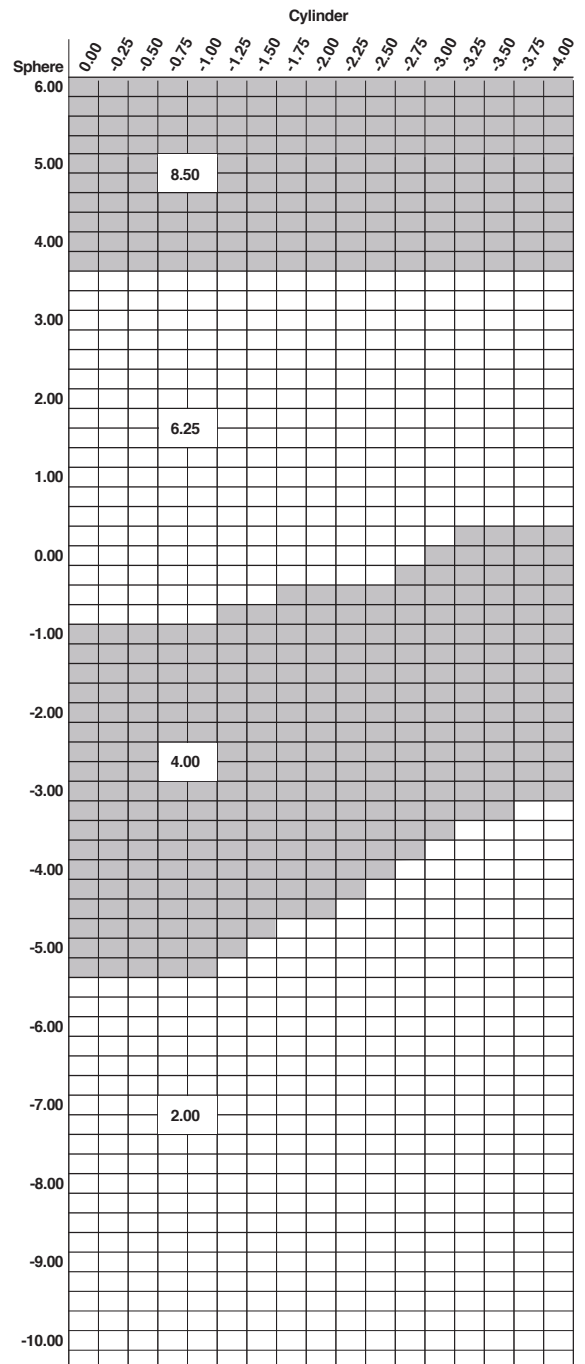

# Product Reference Guide

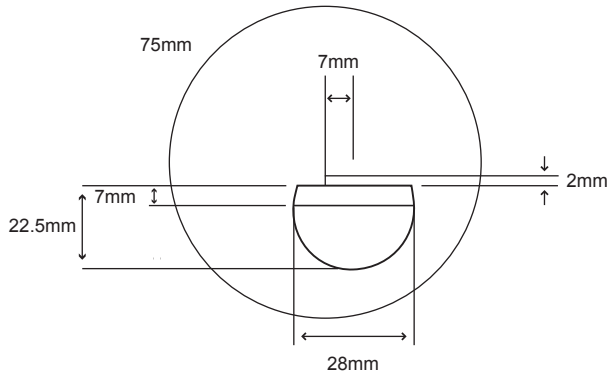

## Base Curve Selection

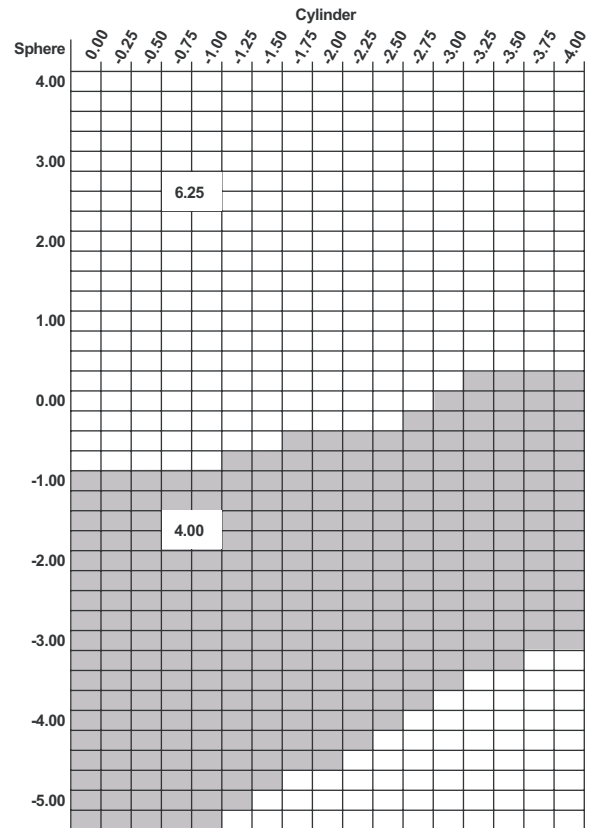

## Polycarbonate

### Coppertone® Prescription Polarized Trifocal


75mm Polarized 7x28 Trifocal (HEV Gray, HEV Brown)

Base Curve	True Curve (1.530)	Front Radius	50mm SAG*	Back Curve	Back Radius	CT (mm) Nominal	ET (mm) Nominal	Segment Drop	Location Inset
4.00	4.07	130.221	2.422	-6.25	-84.800	8.8	12.0	2.0 D	7.0 In
6.25	6.33	83.728	3.819	-6.25	-84.800	8.6	8.5	2.0 D	7.0 In


Base Curve	HEV Gray	HEV Brown
4.00	+2.00 to +3.00	+2.00 to +3.00
6.25	+2.00 to +3.00	+2.00 to +3.00

\*Utilized for power calculation only, not for thickness calculation.

# Product Reference Guide

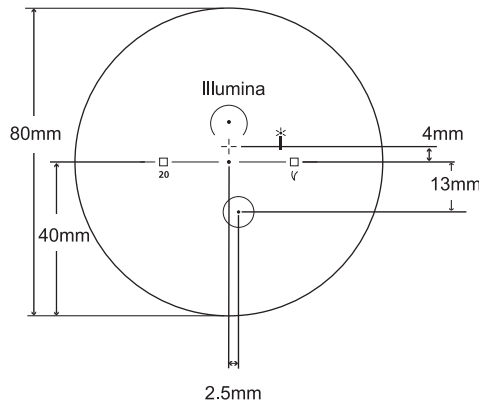

## Base Curve Selection

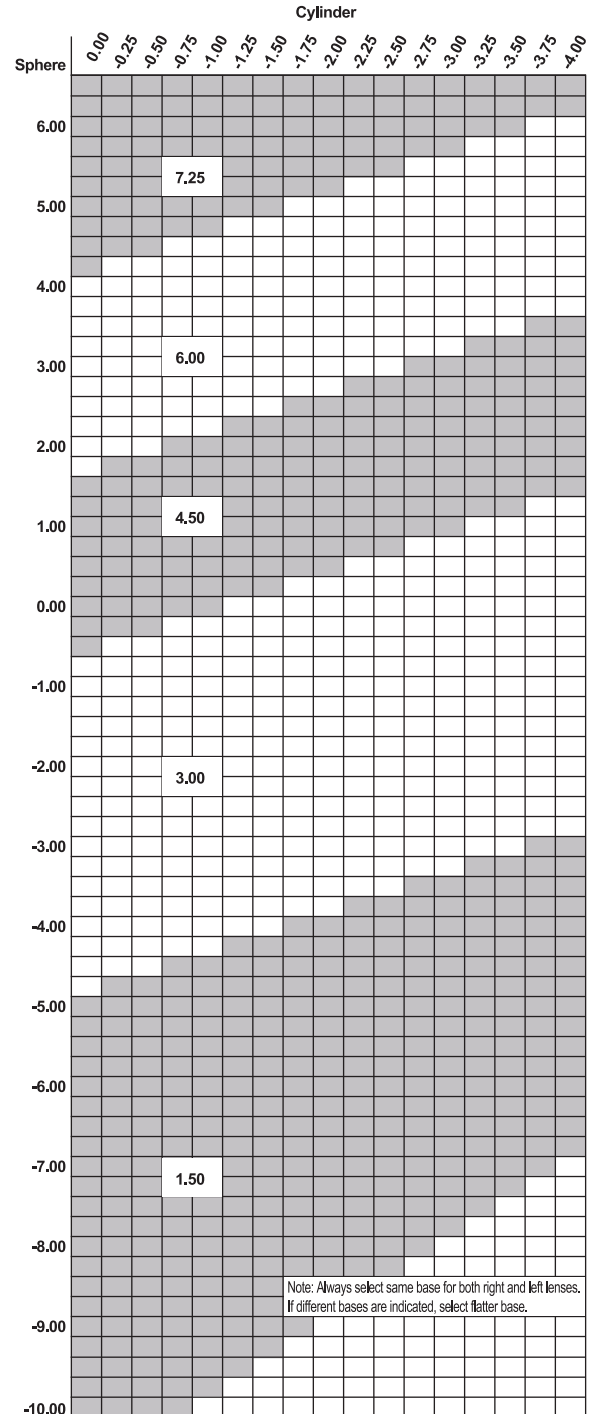

## Polycarbonate

### Illumina® Coppertone® Polarized Progressive Addition Lenses


80mm Polarized Progressive (HEV Brown, HEV Gray)

Base Curve	True Curve (1.530)	Front Radius	50mm SAG*	Back Curve	Back Radius	CT (mm) Nominal	ET (mm) Nominal	MRP Location
1.50	1.61	329.192	0.945	-6.25	-84.800	8.7	15.5	4.0 A 0.0 In
3.00	3.03	174.917	1.800	-6.25	-84.800	8.6	13.0	4.0 A 0.0 In
4.50	4.50	117.777	2.684	-4.53	-116.997	10.7	9.5	4.0 A 0.0 In
6.00	6.00	88.333	3.612	-6.25	-84.800	9.7	9.0	4.0 A 0.0 In
7.25	7.25	73.103	4.408	-7.50	-70.667	10.8	10.5	4.0 A 0.0 In


Base Curve	HEV Brown	HEV Gray
1.50	+0.75 to +3.50	+0.75 to +3.50
3.00	+0.75 to +3.50	+0.75 to +3.50
4.50	+0.75 to +3.50	+0.75 to +3.50
6.00	+0.75 to +3.50	+0.75 to +3.50
7.25	+0.75 to +3.50	+0.75 to +3.50

\*Utilized for power calculation only, not for thickness calculation.

# Product Reference Guide

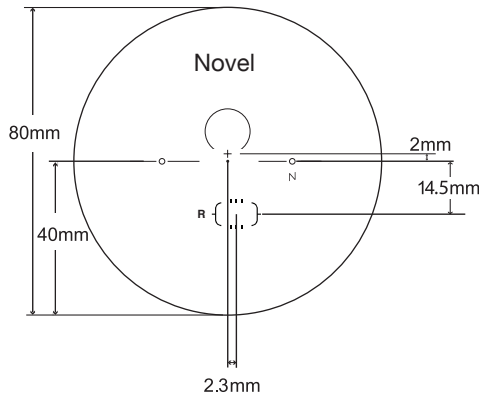

## Base Curve Selection

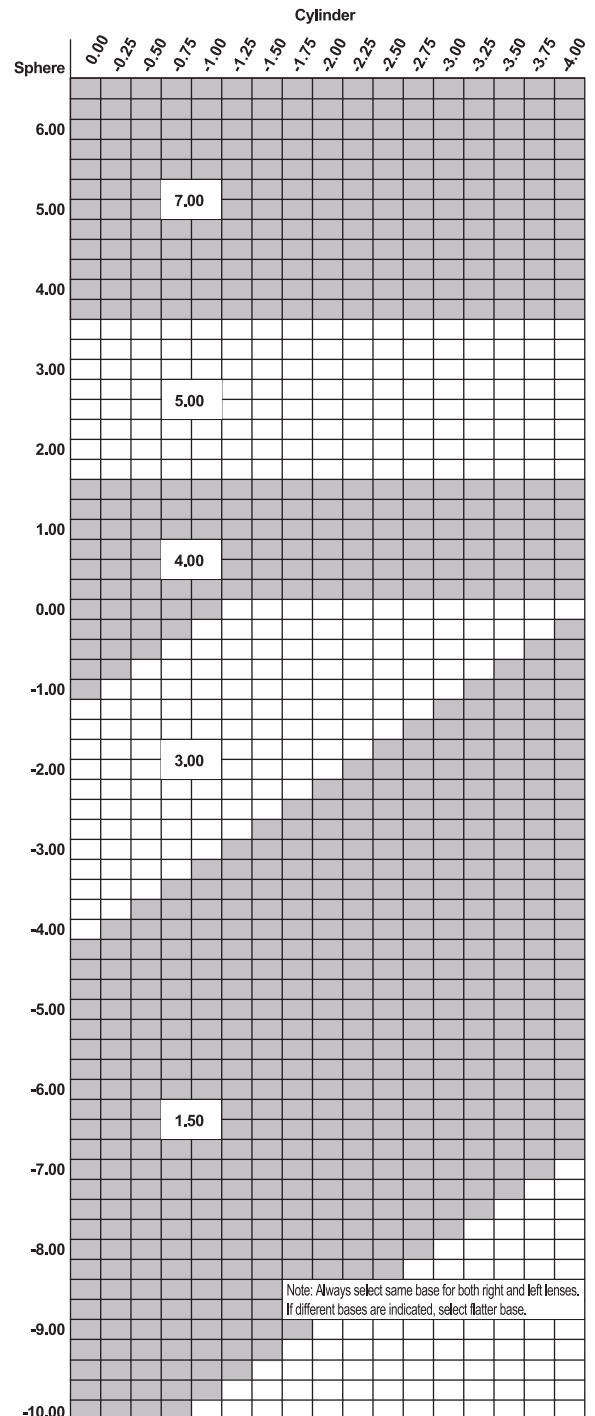

## Polycarbonate

### Novel® Coppertone® Aspheric Progressive Addition Lenses

80mm Polarized Progressive (HEV Gray, HEV Brown)



Base Curve	True Curve (1.530)	Front Radius (mm)	50mm SAG*	Back Curve	Back Radius (mm)	CT (mm) Nominal	ET (mm) Nominal	MRP Location
1.50	1.31	404.58	0.773	-7.50	-70.67	7.0	16.9	2.0 A 0.0 In
3.00	2.73	194.14	1.616	-6.25	-84.8	7.5	12.9	2.0 A 0.0 In
4.00	4.15	127.71	2.471	-6.25	-84.8	8.0	11.2	2.0 A 0.0 In
5.00	5.37	98.70	3.219	-6.25	-84.8	10.1	11.3	2.0 A 0.0 In
7.00	6.90	76.81	4.182	-7.50	-70.65	10.0	11.0	2.0 A 0.0 In

Base Curve	HEV Gray	HEV Brown
1.50	+1.00 to +3.00	+1.00 to +3.00
3.00	+1.00 to +3.00	+1.00 to +3.00
4.00	+1.00 to +3.00	+1.00 to +3.00
5.00	+1.00 to +3.00	+1.00 to +3.00
7.00	+1.00 to +3.00	+1.00 to +3.00

\*Utilized for power calculation only, not for thickness calculation.

# Product Reference Guide

## Base Curve Selection

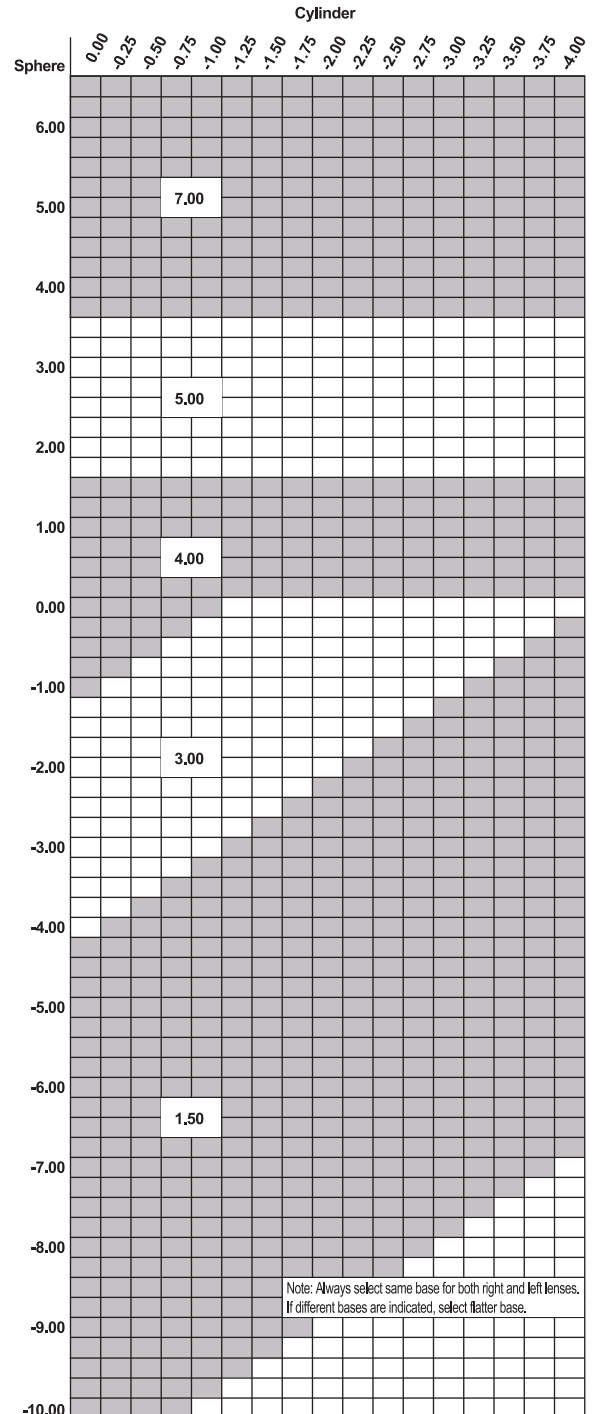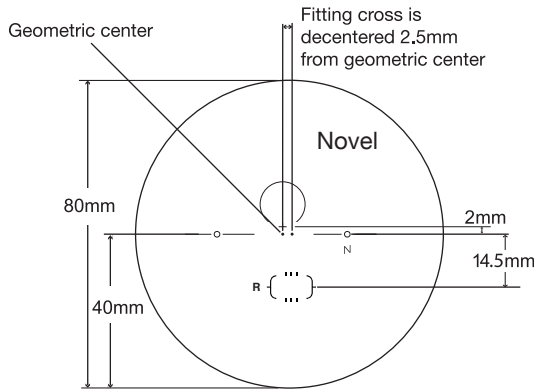

## Polycarbonate

### Novel® Coppertone® Aspheric (Decentered) Progressive Addition Lenses

80/85mm Polarized Progressive (HEV Gray, HEV Brown, HEV Green)

Base Curve	True Curve (1.530)	Front Radius (mm)	50mm SAG*	Back Curve	Back Radius (mm)	CT (mm) Nominal	ET (mm) Nominal	MRP Location
1.50	1.31	404.58	0.773	-7.50	-70.67	7.0	16.9	2.0 A 2.5 In
3.00	2.73	194.14	1.616	-6.25	-84.8	7.5	12.9	2.0 A 2.5 In
4.00	4.15	127.71	2.471	-6.25	-84.8	8.0	11.2	2.0 A 2.5 In
5.00	5.37	98.70	3.219	-6.25	-84.8	10.1	11.3	2.0 A 2.5 In
7.00	6.90	76.81	4.182	-7.50	-70.65	10.0	11.0	2.0 A 2.5 In

Base Curve	HEV Gray	HEV Brown	HEV Green
1.50	+1.00 to +3.00	+1.00 to +3.00	+1.00 to +3.00
3.00	+1.00 to +3.00	+1.00 to +3.00	+1.00 to +3.00
4.00	+1.00 to +3.00	+1.00 to +3.00	+1.00 to +3.00
5.00	+1.00 to +3.00	+1.00 to +3.00	+1.00 to +3.00
7.00	+1.00 to +3.00	+1.00 to +3.00	+1.00 to +3.00

\*Utilized for power calculation only, not for thickness calculation.